## Invivo Workspace Web Viewer 1.5

US: Sept. 4<sup>th</sup>, 2025

<https://learn.osteoidinc.com/invivo-user-manuals>

Anatomage, Inc.  
3350 Scott Blvd. Bldg. #29  
Santa Clara, CA 95054, U.S.A.  
Tel: +1 (408) 333-3484  
Email: support@osteoidinc.com  
www.osteoidinc.com

(01)00852060007105(10)1.5.2

Freyr Regulatory GmbH  
Marie-Curie-Straße 8,  
D-79539 Lörrach, Germany  
Tel: +49 618 170 79007  
www.freyrsolutions.com

### EU:

PaloDEx Group Oy  
Nahkelantie 160, P.O. Box 64  
FI – 04301  
Tuusula, Finland  
Tel: +358 10-270-2000  
regulatory@dexis.com  
www.palodexgroup.com

### Australia Sponsor (TGA)

Henry Schein Halas  
Locked Bag 5003  
Alexandria, NSW, 2015  
customer.care@henryschein.com.au  
Tel: +61 1300-65-88-22

### South Korea

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Indications for Use	<p>Invivo Workspace Web Viewer is a software application used for the display and 3D visualization of medical image files from scanning devices, such as CT, MRI, or 3D Ultrasound. It is intended for use by radiologists, clinicians, referring physicians, and other qualified individuals to retrieve, process, render, review, store, print, assist in diagnosis and distribute images, utilizing standard PC hardware. Additionally, Invivo Workspace Web Viewer is a preoperative software application used for the simulation and evaluation of dental implants, orthodontic planning, and surgical treatments.</p> <p>This device is not indicated for mammography use.</p>
Minimum System Requirements	<p>HTML-5 Compatible Web Browser, such as:</p> <ul style="list-style-type: none"> <li>• Mozilla Firefox 92.0</li> <li>• Google Chrome 94.0</li> <li>• Microsoft Edge 14</li> <li>• Apple Safari 15</li> </ul>
<div style="text-align: center;">  </div> <p>Safety Warnings and Precautions – Refer to ANA003 Invivo Workspace Web Viewer 1.5 Instructions for Use</p>	<ul style="list-style-type: none"> <li>• The software provides tools for dental prosthetics creation and design but is dependent on the user to determine and use the appropriate parameters. Incorrect parameters may affect the quality of the final milled products or otherwise to surgical delays or complications.</li> <li>• Incorrect image orientation may cause surgical delays or complications. The software orients the scan based on the inherent scan information and clearly labels the directionality. However, the software should only be used by licensed clinicians familiar with dental and medical imaging who can best identify any discrepancies with scan orientation in the software.</li> <li>• Please ensure all hardware devices with the software are password-protected from unauthorized use and all patient information is secure.</li> <li>• Implant planning must be performed by a trained healthcare professional for proper implant placement relative to the patient's anatomy. Implant angulation, divergence, location, and depth should abide by the implant manufacturer's instructions for use and placement parameters.</li> <li>• Invivo Workspace Web Viewer's measurement functionality has been verified to be accurate up to +/- 0.10 mm. However, measurement accuracy depends on the image data and the hardware scanner that generated the image data. The measurement cannot be more precise than the resolution of the image. Software reports the value based on user-picked points. Due to the nature of medical imaging, the boundary is not always well defined. The apparent boundary depends on the current brightness and contrast setting. The boundary may shift as the user makes adjustments to brightness and contrast. The user must understand the limitation of the measurement value before applying to the patient. Any measurement that is incorrect can lead to surgical complications if diagnosis, treatment plans and/or actual treatment is based on the incorrect measurements. It is critical for the end user to learn how to perform measurements correctly and employ proper usage of all measurement tools. If you notice any inconsistencies or software problems with measurements, or have further questions or concerns about using measurement tools correctly, please contact us at (408) 333-3484 or email us at support@osteoidinc.com.</li> <li>• Images may be resampled for browser display optimization. Resampled images may not be the same size and resolution or contain the same details as the original scan and are indicated in the rendering window</li> <li>• Any nerve that is traced in a way that does not conform to the actual pathway of the nerve can lead to surgical complications if diagnosis, treatment plans and/or actual treatment is based off of the incorrect tracing. It is critical for the end user to learn how to properly perform nerve tracings correctly. If you notice any inconsistencies or software problems with nerve tracing or have further questions</li> </ul>

	<p>or concerns about nerve tracing, please contact Osteoid support at (408) 333-3484 or email us at support@osteoidinc.com</p> <ul style="list-style-type: none"><li>• Any implant planning that is performed incorrectly in any aspect including but not limited to implant location, orientation, angle, diameter, length, and/or manufacture can lead to surgical complications if diagnosis, treatment plans and/or actual treatment is based off of the implant planning error(s) in question. It is critical for the end user to learn how to use the implant treatment planning tools correctly. If you notice any inconsistencies or software problems with implant planning or have further questions or concerns about correct utilization of implant planning, please contact Osteoid support at (408) 333-3484 or email us at support@osteoidinc.com</li><li>• Soft tissue prediction is an approximation and may not reflect actual results.</li></ul>
--	---

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